



Universal Printed
Materials Polishing Kit
#SR01

What You Will Need:

Universal Printed Materials Polishing Kit

The following workflow includes instruments included in the Universal Printed Materials Polishing Kit (Order No. SR01) developed by Meisinger & SprintRay.

		Photo	Item Number
OPTIONAL:	PRE-POLISH Pre-polish occlusal & broad surfaces with the twist polisher. Use light pressure and a maximum RPM of 7,000. Usage Tips: Use the Pre-Polish system to remove and smooth harpoon support attachment points.		DPP07-170-RA • Polish most areas
			DPP05-140-RA • Polish very tight spaces, interproximal areas
STEP 1:	POLISH Polish with twist polishers for occlusal surfaces and broad areas. Use ultra-light pressure and a maximum RPM of 7,000.		DPP08-170-RA • Polish most areas
			DPP06-140-RA • Polish very tight spaces, interproximal areas
STEP 2:	BUFFING Final shine with the cotton buff wheel.		150-220-HP • Final shine • No paste necessary